## FR

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## GB

**Oxyclean** is a high-quality and user-friendly care system without preservatives.

For the cleaning, disinfection, cleaning of protein and lipid deposits and storage.

For all types of contact lenses including silicone hydrogels.

### Composition:

The preservative-free, aqueous solution contains: hydrogen peroxide 3%, phosphonic acid (stabiliser), sodium chloride, phosphate (buffer system).

Disinfecting and neutralising effect in a single action

**Oxyclean** system offers a safe and easy way to care for your contact lenses. This **Oxyclean** system contains no preservatives.

**Oxyclean** system is for all types of contact lenses including silicone hydrogels lenses.

The hydrogen peroxide disinfect and clean the lenses. The neutralising disc breaks down the hydrogen peroxide into water and oxygen, leaving a saline solution. After the neutralisation procedure is complete, contact lenses can be inserted directly into the eyes.

**Oxyclean** system can also be used in combination with protein removal tablets such as Enzyme-purification tablets.

### Instructions for use:

**Ensure the neutralising disc is attached before use. Oxyclean solution and Oxyclean lens case with the neutralising disc must always be used together.**

Wash your hands thoroughly before removing your contact lenses from your eyes or the holder.

Dry your hands with a clean towel.

Place your left contact lens in the holder cup marked 'L'. Close the cup.

Place the right contact lens in the 'R' cup and close the cup.

Thoroughly rinse the lenses in the baskets with **Oxyclean** for 5 seconds.

Fill the **Oxyclean** lens holder with **Oxyclean** liquid up to the line. NB: Make sure not to use too little or too much liquid. In case of overfilling, the liquid can start foaming and overflow; in case of too little, there is a risk of incomplete disinfection. Use only fresh liquid for each disinfection cycle. Place the lid on the lens holder and screw to tighten. Do not shake the lens holder. Shaking can cause hydrogen peroxide remains to attach to the bottom of the lid. These remains can cause a burning or smarting sensation in contact with your eyes. In that case, immediately rinse the eyes with plenty of clean water.

Then leave the lenses during at least six hours - or preferably all night - in the liquid.

Remove your lenses from the lens holder. The lenses can now be inserted and worn. It is recommended to rinse the lenses with saline liquid before inserting them.

Always dispose of the used liquid immediately. Then rinse the lens holder and platinum disc thoroughly with saline liquid; never use tap water. Rinsing of the disc is also necessary to remove waste deposits that if left on the disc can influence the neutralisation of the liquid. Allow the lens holder and discs to dry in the open air (do not dry using a cloth).

### Warnings:

Keep out of the reach of children.

Store between 4°C and 25°C (40°F and 77°F).

In case of eye irritation, discontinue use and consult your doctor/contact lens specialist.

Make sure the guarantee seal is intact.

Use within 3 months of opening.

Beware not to insert contact lenses into your eyes if they have been removed from **Oxyclean** liquid that is not fully neutralised. Un-neutralised **Oxyclean** liquid is a disinfectant that can cause a burning and smarting sensation in the eyes.

Do not use after the expiry date.

NB: Take a new platinum neutralisation disc after each used bottle.

Avoid regular finger contact with the platinum disc. Fingers can leave grease on the platinum disc, reducing its neutralising effect.

The neutralised solution in the lens holder contains no preservatives. If the contact lenses are left in the solution for longer than 24 hours, it is recommended to disinfect them again.

If you are prescribed eye medication, consult your optometrist or contact lens specialist before using the medication in combination with contact lenses.

In case of the accidental ingestion **Oxyclean** liquid, drink lots of water and consult your physician.

### Side Effects:

Do not use **Oxyclean** if you are allergic to one of the ingredients.

Never use **Oxyclean** for heat disinfection.

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